


DHCP / DNS Error

1. Open your Smartphone Wi-Fi Settings and copy the relevant information (IP Address, Subnet Mask, Router, DNS).

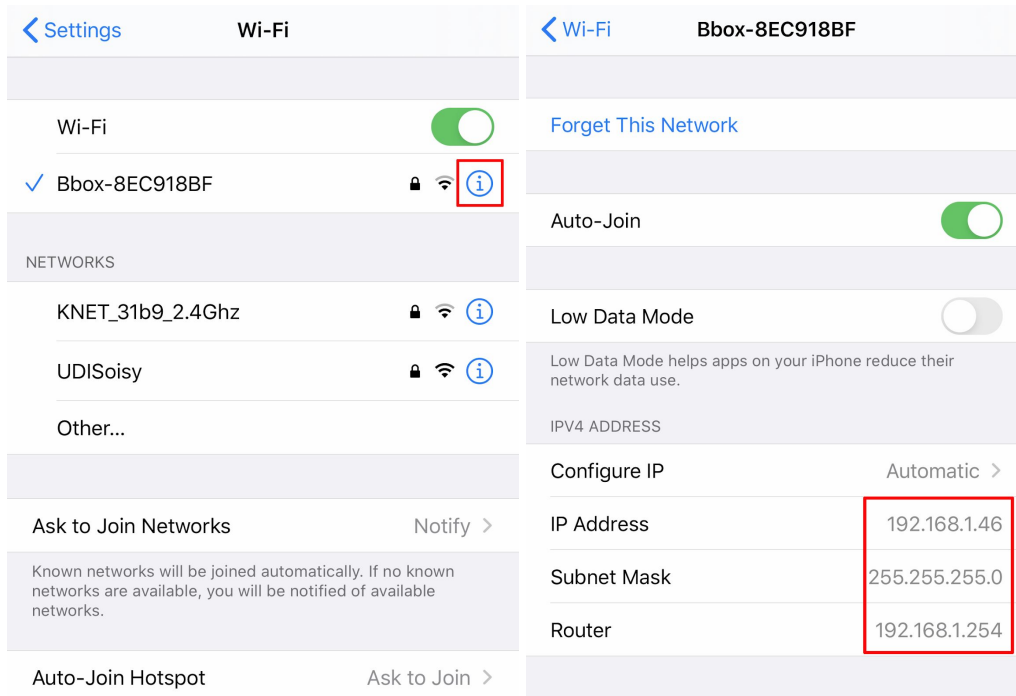

2. When you install your scale, tap **Quick setup** to use the same Wi-Fi network as your mobile device or **Setup another network** to use another one.



3. If the pop-up **Share Wi-Fi Settings** appears, tap **Ignore**.

4. Enable the **Manual Configuration** . To do so, tap **Configure IP > Manual**.

The screenshot shows the iPhone Wi-Fi settings for a network named 'Bbox-8EC918BF'. On the left, the 'Configure IP' option is highlighted with a red box, showing 'Automatic' as the current setting. On the right, the 'Configure IPv4' screen is shown with 'Manual' selected and highlighted with a red box. Below this, the 'MANUAL IP' section contains the following fields: IP Address (192.168.1.250), Subnet Mask (255.255.255.0), and Router (192.168.1.254).

5. Enter the **first 7 numbers** of the IP address and add ".250".

This screenshot is identical to the previous one, but with red boxes highlighting the first seven digits of the IP address (192.168.1), the full Subnet Mask (255.255.255.0), and the Router address (192.168.1.254) in both the 'Configure IP' and 'MANUAL IP' sections.

6. Copy and paste the numbers of the Subnet Mask, the Router, and the DNS.